

## How to see your Impedance on the BrainMaster Training Screen

**Make sure your Atlantis is plugged in but you do not need to have a cable with electrodes attached to complete this procedure.**

- 1) Open BrainMaster
- 2) Click on **Folder Selections**
- 3) Scroll down and Select **"Test"**
- 4) Click **OK**
- 5) Click on **View or Change Settings**
- 6) Click on **Data Channels**
- 7) Click on **Atlantis Hardware Control**
- 8) In the *Hardware Emulation Mode* section - click on **Full Atlantis**
- 9) Click **OK**
- 10) Click **OK**
- 11) Click on **Session Control**
- 12) Click on **Training**
- 13) Click on **OK**
- 14) Click on **Use these Settings**
- 15) Click on **Run the Next Session**
- 16) Click on **Data**
- 17) Click on **Atlantis Setup** (very bottom of list)
- 18) Click on **Check Module**

19) When a window appears with master30, a yellow caution triangle with exclamation mark in it and a bunch of letters and numbers appears. Click on **Ok**

20) Another window pops up that will say something to the effect of “*Current revision and a # and text that reads “An update is available for this device. Do you wish to update this device now?”*” Click on **YES**

21) Read the next window and then click **OK**

22) Watch the status line while it updates. When it is done another window will pop up that says: “*Programming completed successfully. Device will now reset itself.*” Click **OK**

23) Click **CLOSE**

**You are now ready to run the session with viewing impedance in the upper left corner of the training Screen**

**When you go to run the session you will see the impedance reading on the upper left hand corner of the training screen where you used to see text, i.e., *Please perform task... Etc.***

**IMPORTANT NOTE: You need to repeat steps 1- 13 *ONLY* – DO NOT perform steps for each client folder if you want to have the Impedance measures appear on the top of the training screen.**

Copyrighted Instructional Text for BrainMaster by Robert E. Longo ©2009