# BrainMaster<sup>™</sup> System Type 2E and 2EW Modules & Software for Windows<sup>™</sup>

## **Review Screens**

"Breview.exe"

© 1995 - 2004 BrainMaster Technologies, Inc., All Rights Reserved

"BrainMaster" and "From the Decade of the Brain" are trademarks of BrainMaster Technologies, Inc. "Windows" is a trademark of Microsoft Corp.

## BrainMaster System Review Screens ("Breview.exe")

1.1 INTRODUCTION	3
<b>1.2 THE SESSION FILE</b>	3
1.3 STARTING THE REVIEW PROGRAM	3
1.4 VIEWING ALL SESSIONS	4
1.4.1 ALL SESSIONS BY SESSION	5
1.4.2 ALL SESSIONS BY TYPE	6
1.5 SELECTING A SESSION	7
1.6 CHANGING SETTINGS	7
1.7 REVIEWING A SESSION	9
1.7.1 SINGLE SESSION GRAPHICAL DISPLAY	9
1.7.2 SINGLE SESSION WITH THRESHOLDS	10
1.7.3 SINGLE SESSION WITH PERCENT OVER THRESHOLD	11
1.7.4 SINGLE SESSION TEXT FORMAT	12
1.7.5 SINGLE SESSION REPORT FORMAT	13
1.8 EXCEL FILE IMPORT	14
1.8.1 FILE IMPORT PROCEDURE	14
1.8.2 EXCEL TABLE EXAMPLE	15
<b>1.9</b> QUICK-FILE (MINI-Q) FILE EXAMPLE	16
1.10 SAMPLE SUMMARY FILE	17

### 1.1 Introduction

This manual describes the use of the session summary and review screens for the BrainMaster system. This program is **"BReview.exe,"** and it interfaces to the system by reading the standard files written into the directory \brainm.20\studies\[userid]. The programming and file interfaces are documented, to facilitate the development of user-developed and third-party software and applications.

## 1.2 The Session File

The BrainMaster automatically maintains "Summary" files in the current directory. This file contains a textual summary of the EEG component values, their means, and standard deviations for 1-minute intervals, or whenever data collection is started and stopped. This file contains time-stamps with each record. In addition, when any of the number keys (1 through 9) are pressed, the file posts the exact time and date, and records the number. This allows the user to save time-markers for important events. By making a standard use of the numbered markers, up to 9 different types of events can be accurately time-logged.

This file is restarted each time the Training Screen is run. These files are named e.g. "sum00001.bsm," "sum00002.bsm," and so on.

An example of a session file is included in a later section of this manual.

#### 1.3 Starting the Review Program

In order to view the session files written out by the system, do the following:

Using the BrainMaster "Bsetup" program, click on the button labeled "Review Session Results" that is on the main Bsetup Home Screen:

🔯 Setup/Home Screen for BrainMaster 2.5SE	
Welcome to BrainMaster 2.5SE 🛛 🖾 뜣 🞑 🥌	LOGIN OK: SN: 40002 UNLIMITED USE
Current Trainee/Study Folder:	CLINICAL LICENSE
Trainee ID: Test	Login to BrainMaster
Trainee Standard Test Study Name:	Folder Selections
Comment: standard Next Session Number: Total Sessions Available:	Run The Next Session
1 40   Training screen is currently: Not Running	View or Change Settings
Exit BrainMaster Product Manuals	Review Session Results

It is also possible to start breview by double-clicking one one of the session files the directory \brainm.20\studies[trainee\_id] in Windows (using, e.g. "My Computer" or "Windows Explorer"). This will start Breview, and automatically load the referenced file, as shown in the example below:

🚞 TEST					<u>- 🗆 ×</u>
<u>File E</u> dit <u>V</u> iew	F <u>a</u> vori	tes <u>T</u> ools	Help		
G Back 👻 🕥	- 💋	Sear	rch 😥 Folders 🛛 🔓	🕻 🎯 🗙 🦷 🖇	A <u>d</u> dress
Name 💌		Size	Туре	Date Modified	
🗐 xlf00001.txt		1 KB	Text Document	2/26/2004 11:32 AM	
🔊 xlf00001.csv		1 KB	Microsoft Excel Com	2/26/2004 11:32 AM	
💐 sum00001.bsm		2 KB	BSM File	1/17/2004 6:01 PM	
🕖 settings.txt		1 KB	Text Document	1/17/2004 8:13 PM	
🔟 SETTINGS	Type:	BSM File		12/12/2003 11:14 AM	
🔊 qat00001.csv	Date N	lodified: 1/17	2004 6:01 PM Com	2/26/2004 11:32 AM	
🔟 NAME	Size: 1	.78 KB		12/13/1996 8:03 PM	
🗖 email.set		1 KB	SET File	12/12/2003 11:13 AM	
E0690200.002		13 KB	002 File	9/17/1997 4:24 PM	
E0520100.001		1 KB	001 File	7/15/2000 8:47 AM	
E0510100.001		2 KB	001 File	7/15/2000 8:46 AM	
🗖 E0000000.D01		123 KB	D01 File	6/11/1995 8:20 AM	
🗖 CUR_SESS		1 KB	File	4/30/2002 6:43 AM	
🗖 breview.cfg		1 KB	CFG File	2/26/2004 11:32 AM	

The Review screen will appear. If a current session and current study are defined by the system, that session will appear on the screen. If there is a current study but not a current session, Breview will default to displaying all sessions.

## 1.4 Viewing All Sessions

The BReview screen should be visible. It will automatically load the latest, or the current seession, from the current trainee (the last one selected). You may press **"Show All"** to display a bargraph summary of all sessions for the current user. The display format will depend upon the current settings as described below.

#### **1.4.1** All sessions by Session



The screen display above shows all sessions plotted by session.

The system looks for the sessions in order, starting with session 1, and continuing until a session is not found. Therefore, **it is necessary to have all sessions in order, starting with session 1**, for this display to show the sessions properly. Sessions are normally automatically numbered by the "Master.exe" program.

In the current release, the four components initially shown are theta, alpha, low beta, and beta, and are labeled "t", "a", "l", and "b". These are the most common EEG components used for training, and new studies will default to this selection.. It is possible to view the other components (delta, hi beta, gamma, or user) by means of the 'settings' screen, which is described in further detail below.

If you press the Print button, the printer will print out a set of bar graphs, the same as are shown on the display. Printing will usually default to "portrait" mode. If you prefer a "landscape" (horizontal) rather than a "portrait" (vertical) mode printout, you will need to select this option in your printer settings, using the Windows printer control panel that appears when you request your printout. These are different for every printer, but the selection should always be available in one of the control tabs in your printer control.

## 1.4.2 All Sessions by Type

If the current settings have been modified to show all sessions by type, the following display format will result:



### 1.5 Selecting a Session

Press "Select Session" to get a popup box to select the trainee or session to be reviewed.

Select Session	×
Trainee:	Session:
[HEGTEST] ▲ [helga] [HORNDAVI.x [horndavi] [Is-f4] [PAT1] TEST] ▼	SUM00001.bsm SUM00002.bsm SUM00003.bsm SUM00005.bsm SUM00006.bsm SUM00006.bsm
Path: c:\brainm.20\st Session: Text OK	udies\test Cancel

If you select only a trainee from this popup, then press "OK", it will display all sessions for that trainee. If you select a session, it will display the session data. Once you have selected a trainee and session to view, and returned to the main screen, you can still press "All Sessions" to view all sessions for that trainee.

Selecting a trainee for viewing in this manner does NOT change the "current trainee" for the BrainMaster training system to that user. Thus, it is possible to use the review program to review any trainees, without changing the current trainee for the use of the master training program.

## 1.6 Changing Settings

The Settings button brings up the Settings dialog that allows you to change the display settings for the review display:

BReview Settings		×
Components	Channels	- FullScale
□ Delta □ Beta □ Theta □ HiBeta □ Alpha □ Gamma □ LoBeta □ User □ Aux Format □ Supphies □ Logt	Channel 1	C 5 uV C 10 uV C 20 uV C 30 uV C 50 uV C 100 uV
Single Session Show Threshold Levels Show % Time Above Thre Show Modal Frequency Show Standard Deviation Show Phase	Shold Cancel	© 200 uV sions ow By Type ow By Session urce © Digital Filters

This dialog allows you to select the following:

- Components to be viewed
- Channels to be viewed (and coherence if 2-channel data)
- Variables (threshold, percent time, modal frequency, standard deviation, phase) to be viewed
- Full Scale display range in microvolts (or percent if FFT, percent time, coherence, phase, Hz if frequency)
- Format (graphics, text, or report)
- All Sessions format (by type, or by session)
- Data source: FFT (Fast Fourier Transform) or Digital Filters.

Note that FFT data are presented as percent of total energy in the EEG.

Digital Filter data are presented as microvolts of amplitude.

To ensure the correct interpretation when using percent or Hz data, set the FullScale range to 100 before reading values from graphs.

## 1.7 Reviewing a Session

When you select a single session, it will appear on the review screen. When you press "OK", the select box will go away, and the session data will be visible.





If you press the "**Print**" button, the printer will print out a set of line graphs, the same as are shown on the display.

As with the "All Sessions" screen, only the four components, theta, alpha, low beta, and beta are currently shown by default. Other components may be selected by means of the 'settings' button.

If you wish to view the data in more detail, , you may use the **Excel File Import** function described in a following section, to display or process any of the session data in any way you desire.

NOTE: It is recommended that any study have no more than 80 sessions, in order to make it easier to manage the data. If any study will have more than 80 sessions, you should create another trainee ID to accommodate the additional session data.

#### 1.7.2 Single Session with Thresholds

It is possible to display the thresholds, and how they change with time by selecting 'Show Threshold Levels' in the settings screen. An example is shown here:



#### 1.7.3 Single Session with Percent Over Threshold

It is also possible to display the percentage time above threshold for each 60 second time period. In the following example, both the threshold levels and the percentage of time above threshold are being displayed.



## **1.7.4** Single session Text Format

The single session display format may be set to 'text' my means of the Settings button. An example is shown below.

BRevie	w		Liele								
Show A	View Ci All Sel	omponents lect Se	ttings	Redraw	Print	M	lini-Q	Excel	Quit	t	
thoma Text S	s testin ummar	g Sat Ma y: Study:	ay 11 15 1 - FFT I	:18:28 2 Data	002 c:\	brainm.:	20\studie	:s\testth	o2: 22 ∨	alues scale rm	s
Rn/Ch	secs	delta	theta	alpha	lobeta	beta	hibeta	gamma	user	aux	
1/1	60.0	47.48	9.46	12.76	12.08	24.11	138.75	60.87	44.29	0.00	
2/1	60.0	47.75	10.96	14.47	12.42	27.51	153.49	56.75	21.94	0.00	
3/1	60.0	45.39	13.56	19.90	16.62	36.14	171.20	70.91	35.90	0.00	
4/1	60.0	45.26	13.13	18.84	15.85	37.44	171.88	74.76	36.26	0.00	
5/1	60.0	45.99	12.57	19.76	15.59	35.52	172.84	76.05	35.78	0.00	
6/1	60.0	46.78	12.51	16.37	14.33	30.41	168.45	68.36	30.66	0.00	
7/1	60.0	45.95	13.45	19.01	14.90	33.09	171.16	69.74	35.10	0.00	
8/1	60.0	45.21	11.60	18.35	13.61	29.72	169.89	70.15	34.44	0.00	
9/1	60.0	46.23	13.31	15.98	14.32	32.24	170.45	72.15	32.69	0.00	
10/1	60.0	46.36	13.75	18.73	15.70	34.76	174.40	74.99	38.08	0.00	
11/1	60.0	46.46	13.47	20.00	15.62	34.92	174.90	78.26	38.37	0.00	
12/1	60.0	46.57	11.80	14.76	11.87	27.71	167.98	70.00	29.90	0.00	
13/1	60.0	46.90	12.41	16.18	13.51	27.69	167.34	67.28	31.94	0.00	
14/1	60.0	46.02	12.81	17.60	13.34	32.03	171.26	71.59	33.47	0.00	
15/1	60.0	45.75	12.22	19.01	14.15	32.55	169.89	71.34	35.26	0.00	
16/1	60.0	46.27	12.11	16.90	13.76	32.81	171.18	72.98	33.49	0.00	
17/1	60.0	46.25	11.97	17.62	14.53	32.06	168.45	68.69	30.19	0.00	
18/1	60.0	46.52	10.90	14.19	11.17	28.89	166.97	73.44	33.09	0.00	
19/1	60.0	46.46	10.62	14.62	12.39	26.37	169.08	73.03	36.59	0.00	
20/1	60.0	46.18	10.78	13.89	11.43	27.15	166.68	70.30	29.89	0.00	- 00000
21/1	60.0	46.72	10.02	14.62	11.86	26.54	163.76	70.50	32.85	0.00	
22/1	60.0	46.57	11.08	15.27	11.27	26.86	167.17	72.22	32.77	0.00	-

## **1.7.5** Single Session Report Format

The single session display format may be set to 'report' by means of the Settings button. An example is shown below.

BReview										
File Edit View	Componer	nts Help								
Show All S	Select	Settings	Redraw	Prin	t M	ini-Q	Excel	Quit		
Summary Re File: c:\brain	eport: Stu m.20\stu	udy:1 -Se Idies\testtl	ssion Su 102\repor	mmary t.txt	Report				- 12	-
Demographic Trainee N Date: Clinician I	c Inform: ame: D:	ation: thomas te Sat May 1	esting  1 15:18:	28 2002						
Settings: Number of GO: STOP: Filter setti delta: lobeta gamm Artifact thr Number of Dun longti	f channe 1.0 -3 1.0 -3 1: 12.0- 1a: 38.0- reshold: f runs: 5-	ls: 1 lob the: 15.0 theta 15.0 user: 240 10	eta (1.0) ta (5.0) hi : 4.0 -7 15.0- : 30.0- !uV	ibeta (8. 7.0 alı 20.0 hil 35.0 au	0) pha: 8. beta: 20 x: 0.	0 -12.0 1.0-30.0 0 -0.0				
Run lengt	h:	120								
Duration of	of sessio	n:			13	20 secon	ds = 22	minutes 0 s	seconds	
Total poin	ts:				83	4				
Points per	r minute:				37	.9				
l otal % tin	ne over	threshold	(percent )	of timej:	L.1					
104 7	theta 7 25 20	alpna 104 77	10Deta 65.69	Deta 104 77	11Deta	gamma 104 18	104 77			
Total artif	act time:	104.77	03.03	104.77	J2.JJ N	104.10	104.77	0.00		
% time art	ifact:				Ű.	00				
Starting B	aseline	(microvolts	s):		573	1000				
delta	theta	alpha	lobeta	beta	hibeta	gamma	user	aux		
37.87	8.47	12.43	9.28	76.95	84.84	116.78	26.80	0.00		
Ending Ba	seline (I	microvolts	):							
delta	theta	alpha	lobeta	beta	hibeta	gamma	user	aux		
37.87	8.47	12.43	9.28	76.95	84.84	116.78	26.80	0.00		
Starting TI	hreshold	ls (microva	olts):							
delta	theta	alpha	lobeta	beta	hibeta	gamma	user	aux		
	5.00	0.00	1.00	0.00	8.00	0.00	0.00	0.00		
Ending Ih	resholds	s (microvo	itsj:	6						
delta		alpha	Iobeta	Deta	nibeta	gamma	user	aux		
U.UU Crond Mo.	10.90 ana (Dia	U.UU Hal Filtar)	f.40 (miorouv	U.UU	84.40	0.00	0.00	0.00		
Granu Mea	ans (Dig		- (microvi	boto	hihata					
20 17	11.96	14.02	1215	01 47	02 20	120 20	24 59			
Grand Der	centade	e ffrom FF	TJ.TJ D - (nerce	ont of en	or.ju	133.20	24.30	0.00		
dianu PCI delta	thete	a fironi i L alupa	lohete	heta	hibeta	معسمه	licer	3117		
46 32	12 02	16 76	13.65	30.75	167 60	70.65	33.77	0.00		
										-

## **1.8 Excel File Import**

#### **1.8.1** File import procedure

Session data can be read into Excel by selecting the "Quick-File" or the "Excel Table" button at the top of the screen while viewing the session of interest.

NOTE: You must have Microsoft Excel (trademark of Microsoft Corporation) or equivalent software installed on your PC for this feature to work.

When you select one of the Excel options, the system will automatically load the data into Excel, and open a spreadsheet for you to work from.

Excel files have columns and rows. The columns are used to contain the various data for the component bands (delta, theta, etc), and the rows are used to contain the data from successive runs.

Column headings contain labels that identify the type of data contained in that column.

Each record has an entry for the Run number of the respective data ("RUN"). The column NPTS indicates the number of points (seconds) contained in the row's average data. The entries E1 through E5 show the electrode names as entered into the program. When a MINI-Q headbox is used, these are automatically filled in with the correct electrode labels, based on the MINI-Q switch position.

NOTE: The Report, as well as the spreadsheet outputs, will also show any timestamped Notes that you create. You can create a Note at any time, by pressing one of the numeric keys 3, 4, 5, 6, 7, 8, or 9, during a session. The Note will show the exact time and date of the note, and which key was pressed. This can be used to record important time information, by using a convention of your choosing, for the meaning of the key presses. For example, the following convention might be used:

Key	Note Interpretation
Pressed	
3	Eyes Open
4	Eyes Closed
5	Performing Task
6	Eyes Moving
7	Talking
8	Resting
9	Moving

In the 2.0SE release, the notes can be viewed in the Report output, or on the Excel spreadsheets.

#### **1.8.2 Excel Table example**

Shown below is an image of the resulting excel spreadsheet with the labeled fields for the session data. You may now use excel to access any of the data values, plot them, calculate statistics, etc. As shown in the example below, the values for all 8 EEG components are available, and both the "AVG" (average or mean) and "STD" (standard deviation) are available. Values are stored in columns, one for each "run" where a run is 1 minute of the session.

M	licrosoft I	Excel - xlf	00001.c	sv															_ 8	X
	<u>Eile E</u> dit	<u>V</u> iew <u>I</u> ns	ert F <u>o</u> rr	nat <u>T</u> ools	<u>D</u> ata <u>W</u> in	ndow <u>H</u> el	P												_ 8	×
	i 🖉 🚰	86	à 💞	X 🗈	n ·	- 🍓 Σ	E f≈ ≵↓		Aria	1	÷	8 -	B /	<u>u</u> = :		\$	% 💷		• <u>A</u> •	» •
	A1	-	=	c:\brainm	.20\studie	s\BMIR	ROR2\xlft	00001.cs	V											
	Α	В	С	D	E	F	G	Н		J	K	L	M	N	0	Р	Q	R	S	-
1	c:Vorainm.	?0\studies\	BMIRROR	21x1f00001	.csv															-
2	c:\brainm.	20\studies\	BMIRROR	2\SUM0000	01.bsm															
3	RUN	NPTS	E1	E2	E3	E4	E5	DELTAVG	THETAVG	ALPHAVO	LOBAVG	BETAVG	HIBAVG	GAMAVG	USRAVG /	AUXAVG	DELTAVG	THETAVG	ALPHAVGL	л
4	1	60	A1	C3	CZ	C4	A2	4.86	2.06	2.2	1.06	1.11	1.37	1.25	1.08	0	3.05	1.21	0.93	-
6	2	60	A1 A1	0	CZ	C4	A2	3.51	0.71	0.54	0.33	0.32	0.51	0.75	0.41	0	2.42	0.38	0.22	-
7	4	60	A1	63	C7	C4	82	3.57	0.65	0.63	0.55	0.30	0.03	0.04	0.31	0	2.4	0.5	0.20	-
8	5	60	A1	C3	C7	C4	A2	3.4	0.69	0.54	0.34	0.35	0.63	0.89	0.37	0	2.44	0.24	0.17	
9	6	60	A1	C3	Cz	C4	A2	3.53	0.78	0.57	0.3	0.26	0.61	0.8	0.48	0	2.42	0.32	0.28	-
10	7	60	A1	C3	Cz	C4	A2	3.47	0.59	0.62	0.23	0.26	0.7	0.76	0.41	0	2.49	0.27	0.3	1
11	8	60	A1	C3	Cz	C4	A2	3.46	0.63	0.57	0.32	0.29	0.95	0.69	0.4	0	2.37	0.33	0.14	
12	9	60	A1	C3	Cz	C4	A2	3.48	0.64	0.59	0.36	0.4	0.52	0.78	0.33	0	2.39	0.26	0.26	
13	10	60	A1	C3	Cz	C4	A2	3.47	0.64	0.51	0.26	0.26	0.68	0.77	0.32	0	2.35	0.31	0.22	-
14	11	60	A1	C3	Cz	C4	A2	3.42	0.57	0.51	0.34	0.31	0.96	0.84	0.34	0	2.38	0.33	0.23	-
15	12	60	A1	C3	CZ	C4	A2	3.36	0.68	0.53	0.44	0.19	0.65	0.82	0.38	U	2.34	0.4	0.39	-
10	13	60	A1 84	0	CZ	C4	A2	3.4	0.75	0.62	0.37	0.20	0.76	0.04	0.29	0	2.44	0.33	0.34	-
18	14	60	A1	63	C7	C4	82	3.20	0.00	0.51	0.23	0.20	0.0	0.03	0.32	0	2.3	0.20	0.34	-
19	16	60	A1	C3	C7	C4	A2	3.51	0.73	0.50	0.37	0.02	0.89	0.84	0.40	0	2.40	0.27	0.24	-
20	17	60	A1	C3	Cz	C4	A2	3.32	0.71	0.46	0.31	0.24	0.78	0.86	0.26	0	2.38	0.33	0.41	-
21	18	60	A1	C3	Cz	C4	A2	3.3	0.63	0.54	0.38	0.3	0.71	0.95	0.27	0	2.46	0.38	0.45	1
22	19	60	A1	C3	Cz	C4	A2	3.33	0.55	0.5	0.38	0.3	0.62	1.04	0.3	0	2.66	0.43	0.42	7
23	20	60	A1	C3	Cz	C4	A2	3.45	0.65	0.54	0.33	0.31	0.8	0.89	0.41	0	2.52	0.47	0.41	
24	21	60	A1	C3	Cz	C4	A2	3.32	0.78	0.67	0.28	0.32	0.65	0.88	0.29	0	2.6	0.38	0.36	
25	22	60	A1	C3	Cz	C4	A2	3.39	0.65	0.57	0.32	0.32	0.51	0.78	0.44	0	2.54	0.34	0.3	-
20	23	60	A1	C3	CZ	C4	A2	3.51	0.75	0.56	0.35	0.37	0.76	8.0	0.36	0	2.4	0.48	0.35	-
27	24	60	A1 0.1	0	CZ	C4	A2	3.53	0.72	0.54	0.37	0.26	0.69	0.95	0.24	0	2.49	0.39	0.29	-
20	25	60	A1	63	C7	C4	82	3.49	0.02	0.55	0.33	0.32	0.71	0.77	0.40	0	2.52	0.5	0.20	-
30	20	60	A1	C3	Cz	C4	A2	3,58	0.62	0.53	0.47	0.42	0.67	0.81	0.38	0	2.55	0.31	0.35	-
31	28	60	A1	C3	Cz	C4	A2	3.48	0.68	0.52	0.32	0.27	0.72	0.88	0.38	0	2.45	0.33	0.39	
32	29	60	A1	C3	Cz	C4	A2	3.58	0.67	0.52	0.32	0.35	0.87	0.84	0.32	0	2.5	0.36	0.34	
33	30	60	A1	C3	Cz	C4	A2	3.6	0.66	0.45	0.34	0.3	0.87	0.71	0.34	0	2.48	0.4	0.33	1
34	31	60	A1	C3	Cz	C4	A2	3.69	0.79	0.65	0.35	0.35	0.57	0.85	0.39	0	2.53	0.43	0.28	
35	32	60	A1	C3	Cz	C4	A2	3.75	0.8	0.7	0.39	0.33	0.75	0.74	0.46	0	2.59	0.41	0.38	
36	33	18	A1	C3	Cz	C4	A2	3.88	0.73	0.53	0.38	0.49	0.98	0.62	0.28	0	2.59	0.27	0.31	-
3/																				-
30					-							-								-
40	-																			-
14 4		lf00001 /	2						1											Ē
Do-	dv											- I and -		1	5	[		NUM		-
UC0	wy													1.0				IL ACTUAL		

### 1.9 Quick-File (Mini-Q) File Example

You can also select the "Quick-File" option, which combines a wide variety of information in a tabular format, especially useful for mini-assessments, such as those done using the MINI-Q device.

The file can be automatically loaded into excel by selecting the Quick-File button at the top of the screen. An example is shown below:

Mic	Microsoft Excel																	
Eile	Elle Edit View Insert Format Tools Data Window Help																	
l n o	ž 🖪 🙈	l 🚑 🖪	RBC X	🗈 🛍	n + 🎑	$\Sigma f_{*}$	≱i <b>nn</b> 8	?) 🔌 Ari	al	-	8 - 1	<b>B</b> <i>I</i> U		=	\$ %	€≣	- 🕭 - ,	A - 3
	014	- -	=					- · []										_
	1at00001.c	S¥																
	A	В	C	D	E	F	G	H		J	K	L	M	N	0	Р	Q	
	qat00001.c	CSV		00004 h -														
4	C: Vorainm.2	NDTS	CITE	TVDE		THET 0			DET 0	LIDET	COMMO	LICED	TH/01	THAD	TU/DC	01 /DE		
	1	19113	5HE F5	MEAN	55.16	37.11	4LFRA 13.04	EODE1 6.72	10.2	10.37	4.11	03ER 7.36	110AL 2.85	5.52	3.64	4L/DE	-	
5	1	00	F5	MEANE	10.8	22.61	7.72	3.57	7.01	10.37	2.18	7.50	2.03	6.34	2.86	0.08	-	
1 E	1	60	E5	STDDEV	36.2	22.01	6.24	3.47	4.82	4 91	1.86	4 28	4.13	7 44	5.35	1.29		
7	1	60	E5	MODERQ	1.76	4 72	9.51	13.32	18.77	29.85	39.93	10.87	0.5	0.35	0.00	0.51		
8	1	60	F6	MEAN	49.77	36,21	13.34	6,63	9,97	10,55	4.3	6,62	2.72	5,46	3.63	1.34		
9	1	60	F6	MEANF	10.81	23,44	8,27	3,47	8,32	11,06	2.55	2.98	2.83	6,76	2.82	0,99		
10	1	60	F6	STDDEV	32.07	22.86	6.56	3.49	4.76	5.32	1.74	3.43	3.48	6.56	4.8	1.38		
11	1	60	F6	MODFRQ	1.75	4.77	9.47	13.35	18.7	30.11	39.92	10.79	0.5	0.36	0.26	0.51		
12	1	60	F5-F6	COHE	95.48	82.52	66.37	47.8	32.72	24.68	21.12	56.1	1.24	1.73	2.52	2.03		
13	1	60	F5-F6	PHASE	7.88	12.22	18.98	26.63	33.85	36.92	37.78	22.4	0.64	0.46	0.36	0.56		
14	1	60	F5/F6	ASYM	1	0.96	0.93	1.03	0.95	0.93	0.85	1.01	1.03	0.94	1.02	0.98		
15	2	60	F5	MEAN	36.11	27.06	11.62	7.05	9.48	10.95	4.7	7.08	2.33	3.84	2.86	1.23		
16	2	60	F5	MEANE	8.59	20.19	7.63	3.66	9.13	13.29	2.64	2.97	2.65	5.52	2.21	0.84		
17	2	60	F5	STDDEV	26.88	17.95	5.22	3.41	3.11	5.08	2.11	4.51	3.44	5.27	5.78	1.68		
18	2	60	F5	MODFRQ	1.72	4.86	9.51	13.4	19.01	29.76	39.89	10.86	0.51	0.36	0.26	0.5		
19	2	60	F6	MEAN	31.01	28.35	11.79	6.53	7.55	8.91	4.11	7.37	2.4	4.34	3.75	1.56		
20	2	60	F6	MEANE	8.61	23.51	8.5	4.08	8.63	12.22	2.62	3.32	2.76	5.77	2.72	0.98		
21	2	60	F6	STDDEV	22.15	14.55	5.11	2.99	2.5	2.61	1.74	4.45	2.85	4.86	5.82	2.04		
22	2	60	F6	MODFRQ	1.67	5	9.46	13.36	18.76	30.17	39.83	10.87	0.53	0.37	0.27	0.5		
23	2	60	F5-F6	COHE	87.9	82.1	65.2	44.97	19.45	20.68	18.22	65.35	1.26	1.83	4.22	3.35		
24	2	60	F5-F6	PHASE	13.67	15.3	18.55	28.35	37.98	40.1	42.7	20.68	0.82	0.54	0.4	0.49		
25	2	60	F5/F6	ASYM	1	0.86	0.9	0.9	1.06	1.09	1.01	0.89	0.96	0.96	0.81	0.85		
26	3	2	F5	MEAN	20.7	31.1	16.15	10.55	20	24.15	10.85	5.85	1.93	2.95	1.55	0.81		
27	3	2	F5	MEANF	3.6	10.15	6.65	3.55	12.15	26.2	5.05	2.75	1.53	2.86	0.84	0.55		
28	3	2	F5	STDDEV	1.56	12.73	1.63	0.78	3.54	9.83	1.2	2.76	7.83	16.36	3.6	0.46		
29	3	2	F5	MODFRQ	1.45	5.1	9.65	13.45	19.35	29.2	39.6	10.85	0.53	0.38	0.26	0.5		
30	3	2	F6	MEAN	17.6	29.1	11	8.2	16.5	15.2	6.3	4.4	2.65	3.55	1.76	0.67		
31	3	2	F6	MEANF	4.4	15.7	6.9	4.75	12.55	21.3	3.9	2.55	2.28	3.31	1.25	0.55		
32	3	2	F6	STDDEV	12.02	16.83	3.54	7.78	9.05	11.03	4.67	1.27	4.76	2.16	1.86	0.39		
33	3	2	F6	MODFRQ	1.45	5.35	9.4	13.6	19.3	29.6	40.1	10.85	0.57	0.39	0.28	0.49		
34	3	2	F5-F6	COHE	72.5	85	67.5	31.5	33.5	30.5	25	48	1.26	2.7	2.54	2.01		
35	3	2	F5-F6	PHASE	19	16	20	29	32.5	34	40.5	24.5	0.8	0.55	0.49	0.62		
36	3	2	F5/F6	ASYM	0.82	0.65	0.96	0.75	0.97	1.23	1.29	1.08	0.67	0.87	0.67	1		
37	4	59	F5	MEAN	19.52	27.78	13.94	6.12	6.75	4.82	1.24	7.19	1.99	4.54	4.11	2.06		
		st000017		I ANT A A DT			10.00	1.05	7.50		î^	5.00	~ ~ ~ ~	5.00	0.70			M I 🗸
			-	-	-		-	-				1						
Read	У																	

The rows provided in the Quick-File ar as follows: MEAN: mean amplitude from digital filter data MEANF: mean fraction of EEG energy in band, from FFT STDEV: standard deviation of digital filter data MODFRQ: modal frequency (dominant frequency) from FFT

When 2 channels are gathered, the following is also provided: COHE: coherence value (in percent) PHASE: phase value (in percent phase: 0 = inphase, 100 = 180 degrees out of phase) ASYM: asymmetry (from MEANF data)

## 1.10 Use of Auxiliary Review Software

There is the ability to use auxiliary software to review and process data. This software includes NeuroGuide (Applied Neurosciences, Inc) and DNS128 (Digital Clinical Norms, Neurofedback of Southern Vermont, LLC).

To use DNS128, use the "Review SW" control button on the Review Screen:

🔲 BRe	view for B	rainMaste	er 2.5							
<u>Eile E</u> d	lit <u>V</u> iew Co	mponents	Help	1						
Sho	w All 🤤	Select	Settings	Redraw	Print	Quick-File	Excel Table	ReviewS₩	Quit	
10 r 8 -	mike gism Granc Digita Theta Alpha Lobet Beta:	ondi Th I Averag II Filter F : : a:	nu Sep 08 18:3 es, session: 4 lesults: Chan 1 6.27 7.21 6.18 7.90	37:02 2005	c:\brainm.20\ O\studies\gis	studies\gism mmike sumO(	mike: 4 values	scale pk-pk d	ligital filter	Beta Alpha Lobeta
4	<u>0</u>									_
2	<u>.</u>		1		2	5		1		4

Having pressed this button, you will see a "browser" window with sofware shortcuts available:

🗁 ReviewSW	
Elle Edit View Favorites Tools Help	- <b>1</b> 1
G Back - O - p Search - Folders	
Address 🗁 C:\brainm.20\ReviewSW	🖌 🄁 Go
File and Folder Tasks 🛞 🗘 DCN128 DCN128 DD Elder Quick Assessment Files	
Other Places	
Details Shortcut	
Digital Clinical Norms 128 by John Demos Shortcut	

The Digital Clinical Norms 128 shortcut should be double-clicked (or right-clicked and select "Open". You must have Microsoft Office 2003 or later to use DCN128). Once the software

begins the procedures and documention are accessible from within the software, and also from online sources.

To use NeuroGuide, you first must select "Save EEG to Disk" on the Data Channels control of the Main BrainMaster software. Then, you start the NeuroGuide software, and select the EEG records you want to analyze. This procedure is documented in a separate document.

#### 1.11 Sample Summary File

The summary file contains "tagged" fields that can be read by a human, or by a program. To write a program to read in the data, you should set up your software to read in the file a line at a time, and by inspecting the label at the beginning of the field, to determine what it contains. Then, your program can read in and process the data. Note that this file specification is subject to change, and that additional fields may appear. NOTE: second-by-second summary data are also available in this file. Contact BrainMaster Technologies for details.

SUMMARY FOR: bmirror1 FILE: sum00001 BEGIN\_ACQUISITION 1 DATE Sun Apr 14 12:40:18 2002 BEGIN\_PARMS DATE: Sun Apr 14 12:40:18 2002 SESS: 1 NCHANS: 1 NVALS: 8 VALNAMES: DELTA THETA ALPHA LOBETA BETA HIBETA GAMMA USER INTERVAL: 1.0 NINTERVALS\_RUN: 60 FLAG00: 1 0 2 0 0 0 0 0 FLAG01: 0 1 1 1 1 0 0 0 FLAG02: 1 1 0 0 1 0 0 0 FLAG03: 0 1 0 0 1 0 0 0 FLAG04: 0 0 0 0 0 0 0 0 FLAG05: 6 0 0 0 FLAG06: 0 0 0 1 0 0 0 0 FLAG07: 0 0 0 1 0 2 0 MIDI\_VOICE: 68 0 0 MODE01: 0 0 0 0 0 0 0 0 MODE02: 0 0 0 0 0 0 0 0 THRS01: 0 0 0 0 0 0 0 0 THRS02: 0 0 0 0 0 0 0 0 THRSA: 240 50 AUTOTH00: 60 20 10 AUTOTH01: 0 0 0 0 0 10 0 0 AUTOTH02: 0 0 0 0 0 10 0 0 DUR: 0 1.00 FRL001: 4.00 9.50 12.00 15.00 20.00 38.00 7.00 7.00 12.00 15.00 20.00 38.00 40.00 FRHI01: 3.00 9.00 ELEC: A1 Cz A2 -- --RWDSMPS: 60 END\_PARMS REINITIALIZING: BEGIN\_TRAINING 1 DATE Sun Apr 14 12:40:29 2002 BEGIN\_SUMM 1: NPTS: 60 DATE Sun Apr 14 12:41:28 2002 0 0 0 Ω MIN01: 16 0 0 0 MAX01: 68 82 30 40 31 80 80 43 MEAN01: 31.02 18.42 5.68 7.64 10.66 14.92 10.47 7.95 MEANF01: 80.44 19.08 9.14 8.05 13.81 40.12 5.56 8.95

Part Number 531-007 Version 2.5SE 5/30/06

STD01: 9.67 16.65 6.93 8.49 8.68 14.74 16.90 8.05 7680 7680 7680 TIM01: 7680 7680 7680 7680 7680 TOT01: 100 100 100 100 100 100 100 100 PTR: 59 HITS: 0 ARTF: 0 END\_SUMM 1 Etc. etc. etc. **Two Channel Summary:** SUMMARY FOR: bmirror2 FILE: sum00001 BEGIN\_ACQUISITION 1 DATE Sun Apr 14 19:12:27 2002 BEGIN\_PARMS DATE: Sun Apr 14 19:12:27 2002 SESS: 1 NCHANS: 2 NVALS: 8 VALNAMES: DELTA THETA ALPHA LOBETA BETA HIBETA GAMMA USER INTERVAL: 1.0 NINTERVALS\_RUN: 60 FLAG00: 1 0 2 0 0 0 0 0 FLAG01: 0 1 1 1 1 0 0 0 FLAG02: 1 1 0 0 1 0 0 0 FLAG03: 0 0 0 0 1 0 0 0 FLAG04: 0 0 0 0 0 0 0 0 FLAG05: 6 0 0 0 FLAG06: 0 0 0 1 0 0 0 0 FLAG07: 0 0 0 1 0 2 0 MIDI\_VOICE: 68 0 0 MODE01: 0 0 0 0 0 0 0 0 MODE02: 0 0 0 0 0 0 0 0 THRS01: 0 0 0 0 0 0 0 0 THRS02: 0 0 0 0 0 0 0 0 THRSA: 240 50 AUTOTH00: 60 20 10 AUTOTH01: 0 0 0 0 0 10 0 0 AUTOTH02: 0 0 0 0 0 10 0 0 DUR: 0 FRLO01: 1.00 4.00 9.50 12.00 15.00 20.00 38.00 7.00 7.00 12.00 15.00 20.00 38.00 40.00 FRHI01: 3.00 9.00 ELEC: A1 C3 Cz C4 A2 RWDSMPS: 60 END\_PARMS REINITIALIZING: BEGIN\_TRAINING 1 DATE Sun Apr 14 19:12:40 2002 BEGIN\_SUMM 1: NPTS: 60 DATE Sun Apr 14 19:13:38 2002 MIN01: 20 0 0 0 0 0 0 0 340 246 240 MAX01: 413 430 166 66 116 MEAN01: 48.56 20.59 21.97 10.59 11.08 13.69 12.46 10.75 MEANF01: 89.98 19.20 20.54 13.88 16.83 40.41 4.58 11.75 STD01: nan nan nan 30.13 12.46 19.84 17.63 23.88 SIM01: 76.22 3.39 2.63 2.37 1.69 3.73 0.80 3.80 7560 7560 7560 7560 7560 TIM01: 7560 7560 7560 100 100 100 100 100 100 тот01: 100 100 PTR: 59 MTN02: 10 0 0 0 0 0 0 0 225 125 175 250 125 170 110 60 MAX02: MEAN02: 30.53 12.10 9.27 8.75 5.17 8.92 12.32 4.73 MEANF02: 89.34 14.51 12.46 8.17 11.85 30.36 4.49 8.78 nan 19.25 25.89 31.51 17.79 25.16 21.52 11.24 STD02:

Part Number 531-007 Version 2.5SE 5/30/06