

## Installation of Animation Software files


If your PassKey includes access to either the MultiMediaPlayer (or the Animation UnLock software), and you want to use any animations, you will want to load the available animation files. You can either do this now or come back to do it at a later time.

Again, the preferred method to install the animation files is from the Brainmaster website. To find the files, return to this URL: <http://www.brainm.com/software/2.5/> (or insert the installation CD, and go to the main CD display) and scroll down.

Load the animation files you want from the following screen.

Note for CD-method of software installations: the Basic Animations do not fit on the 2.5SE CD and must be loaded from a 2.5Core Software CD.

You should see a screen like this; Click on the word “HERE” to download the desired files.

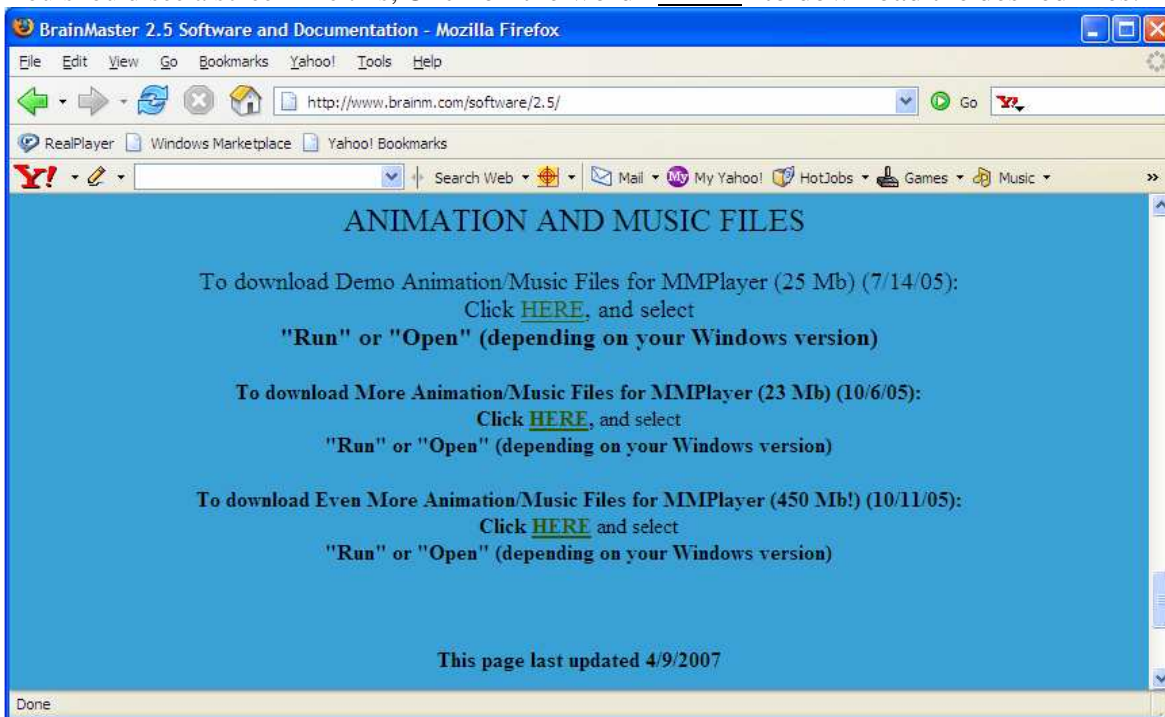

The files will automatically unzip and install onto your computer as soon as you click the word “HERE”.